## **Better Care Better Life**

Safe Injection

Away From Steel Needles

### 江西三鑫医疗科技股份有限公司

Jiangxi Sanxin Medtec Co., Ltd. 地址: 江西省南昌小蓝经济开发区富山大道999号 Location: No.999 Fushan Road XiaoLan Economic Development Zone Nanchang 330200, Jiangxi , P.R. China 电话 (Tel) : 0791-85981988 邮箱 (E-mail) : sales@sanxin-med.com 网址(Web):www.sanxin-med.com



\_\_\_\_\_



# MEDICAL NEEDLE FREE INJECTOR















### [Product Name] Medical needle-free injector

[Product Technical Requirement Number] National Instrument registration 20233140957

### [Production License No.]

Jiangxi Drug Regulatory Machinery Production No. 20150058

### [Registration Certificate Number]

National Instrument Registration License 20233140957

### **Sterilization method**

Dose chamber components and injection devices are sterilized by ethylene oxide.

### **Concise Instructions**

[Scope of Application] The product is used for insulin subcutaneous injection.

### [Validity]

Dose chamber Set and filling device are valid for three years from the date of ethylene oxide sterilization; Needle-free injection thruster and loading boxes are non-sterile with maximum usage of 5000 times.

### **[Transportation Conditions]**

The product should be protected from heavy pressure, direct sunlight and rain and snow immersion during transportation.

### [Storage Conditions]

The product should be stored in a non-corrosive gas and well-ventilated room, and the product should be fully protected, and should be kept away from chlorine-containing disinfection items.

### [Structure and Composition]

The medical needle-free injector is composed of a needle-free injection thruster, a dose chamber set (ampoule bottle), a filling device and a loading box. The needle-free injection thruster and loading box are non-sterile and reusable, and the dose chamber set (ampoule bottle) and filling device are sterilized by ethylene oxide for one-time use.